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English	Ingredients
<i>Please read these instructions carefully before use.</i>	Water, linseed extract*, aloe vera*, locust bean gum*, guar gum*, sodium chloride, xanthan gum, potassium sorbate, citric acid, phenoxyethanol.
menoelle Gel	<i>*Certified organic ingredients. 95% organic.</i>
menoelle Gel is a certified organic moisturising gel for vaginal use. menoelle Gel uses patented technology and combats vaginal dryness with natural, long-lasting rehydration.	International patent protection.
Use	menoelle Gel is a gentle, colourless gel that can be simply applied and distributed with your finger tips (no applicator supplied). Therefore, you can decide yourself, how much and where you wish to apply it. Squeeze the desired amount of gel from the tube and distribute it with your fingertips to the inside walls of your vagina. menoelle Gel can also be used to moisten the lips of the vagina. Product does not come with a reusable applicator.
Frequency of use	Application of 5 ml every two to three days is usually enough to keep vaginal tissues hydrated and comfortable. In cases of extreme dryness, the gel can initially be used more frequently (1x every one to two days), until there is an improvement. For easing pain during sexual intercourse, the gel should be applied before intercourse and again afterwards to provide relief.
Notice	Isotonic*: The isotonic/iso-osmotic property of menoelle Gel is the same as that of human tissue. Why is this balance necessary? Hypertonic products (which generally contain chemicals like glycerine and glycol) can cause vaginal irritation. menoelle Gel 's isotonic/iso-osmotic property protects against irritation. <i>*Further information can be found on our website.</i>
menoelle Gel:	Storage: When not in use, always store the tube with the lid closed. Store menoelle Gel in a cool, dry place at temperatures of between +5 and +30 °C. Protect against direct sunlight. Use before expiry date. Caution: menoelle Gel is neither a spermicide nor a contraceptive. Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice. <i>Not compatible with polyurethane condoms.</i>
menoelle Gel:	Certified organic by: The Soil Association, Bristol, UK. Approved by: The Vegetarian Society.
Distributor	Distributor and import: PhytoLife Nutrition GmbH Winsstrasse 59 10405 Berlin Germany Manufacturer: The YES YES Company Ltd. The Timber Barn, Selborne Road Greatham, Liss, Hampshire GU33 6HG UK
	   1639

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